

**Philips MRI Protocol Dump**  
**Created on**  
2/2/2017 2:59:27 PM  
**Comment**  
Created by ExamCard\_to\_XML with inputs: "E:\Export\CDIP\_HUMAN (Master patch -32 canaux).ExamCard" on system ( :: 192.168.71.10)  
**Software Stream**  
5.1.8.0

---

Expand All | Collapse All  
Hospital (10) (1) (8)  
Hospital (5)  
CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1  
Survey 00:31.4  
T1W\_3D\_TFE\_SENSE2 06:19.9  
DUAL\_SENSE2 05:00.0  
AX\_FLAIR\_SENSE2 03:36.0  
T2\_FFE\_SENSE2 04:13.5  
rs\_fmri 08:53.8  
DTI\_32\_iso 05:50.4  
DTI\_b0\_AP 00:31.1  
Head CNS (3)  
DYN (0)  
SWIp (0)  
FO\_SUP (0)  
Abdomen (1)  
Pelvis (1)  
[B1860723-4F8F-476e-8075-D42C65706693] (0)

Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1 | Survey 00:31.4

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	00:31.4	Nucleus	H1	Scan type	Imaging
Rel. SNR	1.000167	Uniformity	CLEAR	Scan mode	M2D
Act. TR/TE (ms)	11 / 4.6	FOV FH (mm)	250	technique	FFE
ACQ matrix M x P	256 x 128	AP (mm)	250	Contrast enhancement	T1
ACQ voxel MPS (mm)	0.98 / 1.95 / 10.0	stack RL (mm)	50	Acquisition mode	cartesian
REC voxel MPS (mm)	0.98 / 0.98 / 10.0	Voxel size FH (mm)	0.976563	Fast Imaging mode	TFE
Scan percentage (%)	50	AP (mm)	1.953125	shot mode	multishot
TFE shots	2	Slice thickness (mm)	10	TFE factor	64
TFE dur. shot / acq (ms)	1161.8 / 704.0	Recon voxel size (mm)	0.9765625	startup echoes	default
TFE shot interval (ms)	1161.829	Small FOV imaging	no	shot interval	shortest
Min. TI delay	396.7518	Fold-over suppression	no	profile order	linear
Act. WFS (pix) / BW (Hz)	3.104 / 139.9	Reconstruction matrix	256	Echoes	1
Min. WFS (pix) / Max. BW (Hz)	0.560 / 775.0	SENSE	no	partial echo	no
Min. TR/TE (ms)	9.4 / 3.7	+MultiBand SENSE	no	shifted echo	no
SAR / local torso	< 11 %	k-t BLAST	no	TE	in-phase
Whole body / level	< 0.3 W/kg / normal	Stacks	3	(ms)	4.60545
SED	0.0 kJ/kg	current	A	Flip angle (deg)	15
B1+rms / Coil Power	0.75 uT / 10 %	type	parallel	TR	user defined
Max B1+rms	0.75 uT	slices	3	(ms)	11
PNS / level	53 % / normal	slice gap	user defined	Halfscan	no
dB/dt	44.8 T/s	gap (mm)	10	Water-fat shift	user defined
Sound Pressure Level (dB)	17.52768	slice orientation	sagittal	(pixels)	3.5
Boil-off (hPa/h)	2.377374	fold-over direction	AP	RF Shims	fixed
Max Boil-off (hPa/h)	3.987857	fat shift direction	F	Shim	default
Boil-off (hPa)	0.02071577	Slice scan order	default	mDIXON	no
<b>MOTION</b>		Stack scan order	ascend	Fat suppression	no
Cardiac synchronization	no	Move table per stack	no	Water suppression	no
Heart rate > 250 bpm	no	Stack alignment	no	TFE prepulse	invert
Respiratory compensation	no	Stack display order	no	slice selection	no
Navigator respiratory comp	no	PlanAlign	no	shared	no
+RNAV imaging mode	no	REST slabs	0	delay	user defined
Flow compensation	no	Catheter tracking	no	(ms)	800
fMRI echo stabilisation	no	Interactive positioning	no	PSIR	no
Motion smoothing	no	Allow table movement	no	+MP2RAGE	no
NSA	1	<b>OFFC/ANG</b>		+VASO-BOLD	no
<b>DYN/ANG</b>		Stacks	3	MTC	no
Angio / Contrast enh.	no	current	A	T2prep	no
Quantitative flow	no	Stack Offc. AP (P=+mm)	-20	+DANTE	no
Manual start	no	RL (L=+mm)	0	Research prepulse	no
Dynamic study	no	FH (H=+mm)	0	Diffusion mode	no
Arterial Spin labeling	no	Ang. AP (deg)	0	Elastography mode	no
<b>POST/PROC</b>		RL (deg)	0	Transmit channels	both
Preparation phases	auto	FH (deg)	0	SAR mode	high
Interactive F0	no	Free rotatable	no	B1 mode	default
SmartPlan survey	no			SAR Patient data	auto
B0 field map	no			PNS mode	low
MIP/MPR	no			Gradient mode	default
SWIp	no			SofTone mode	no
Images	M, no, no, no				
Autoview image	M				
Calculated images	no, no, no, no				
Reference tissue	White matter				
Recon compression	No				
Preset window contrast	soft				
Reconstruction mode	immediate				
Save raw data	no				
Hardcopy protocol	no				
Image filter	system default				

Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1 | T1W\_3D\_TFE\_SENSE2 06:19.9

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	06:19.9	Nucleus	H1	Scan type	Imaging
Rel. SNR	1.000037	Uniformity	CLEAR	Scan mode	3D
Act. TR/TE (ms)	7.3 / 3.3	FOV FH (mm)	256	technique	FFE
ACQ matrix M x P	256 x 248	AP (mm)	248	Contrast enhancement	T1
ACQ voxel MPS (mm)	1.00 / 1.00 / 1.00	RL (mm)	180	Acquisition mode	cartesian
REC voxel MPS (mm)	1.00 / 1.00 / 1.00	Voxel size FH (mm)	1	Fast Imaging mode	TFE
Scan percentage (%)	100	AP (mm)	1	3D non-selective	no
TFE shots	126	RL (mm)	1	shot mode	multishot
TFE dur. shot / acq (ms)	1861.5 / 1817.1	Recon voxel size (mm)	1	TFE factor	248
Min. TI delay	944.3877	Fold-over suppression	no	3D free factor	no
Act. WFS (pix) / BW (Hz)	1.903 / 228.2	Slice oversampling	default	startup echoes	default
Min. WFS (pix) / Max. BW (Hz)	0.560 / 775.0	RF select. FOS	no	shot interval (ms)	user defined 3000
SAR / local torso	< 9 %	Reconstruction matrix	256	profile order	linear
Whole body / level	< 0.3 W/kg / normal	SENSE	yes	turbo direction	Y
SED	< 0.1 kJ/kg	P reduction (AP)	1	Echoes	1
B1+rms / Coil Power	0.68 uT / 8 %	S reduction (RL)	2	partial echo	no
Max B1+rms	0.68 uT	k-t BLAST	no	shifted echo	no
PNS / level	55 % / normal	Overcontiguous slices	no	TE	shortest
dB/dt	56.1 T/s	Stacks	1	Flip angle (deg)	9
Sound Pressure Level (dB)	13.20022	slices	180	TR	shortest
Boil-off (hPa/h)	3.603582	slice orientation	sagittal	Halfscan	no
Max Boil-off (hPa/h)	5.937897	fold-over direction	AP	Water-fat shift (pixels)	user defined 1.9
Boil-off (hPa)	0.3802395	fat shift direction	F	RF Shims	fixed
<b>MOTION</b>		Multi-chunk	no	Shim	auto
Cardiac synchronization	no	PlanAlign	no	mDIXON	no
Heart rate > 250 bpm	no	REST slabs	0	Fat suppression	no
Respiratory compensation	no	Catheter tracking	no	Water suppression	no
Navigator respiratory comp	no	Interactive positioning	no	TFE prepulse	invert
+RNAV imaging mode	no	Allow table movement	no	slice selection	no
Flow compensation	no	<b>OFFC/ANG</b>		delay	shortest
fMRI echo stabilisation	no	Stacks	1	PSIR	no
Motion smoothing	no	Stack Offc. AP (P=+mm)	-18.11262	+MP2RAGE	no
NSA	1	RL (L=+mm)	-3.655661	+VASO-BOLD	no
<b>DYN/ANG</b>		FH (H=+mm)	34.79839	MTC	no
Angio / Contrast enh.	no	Ang. AP (deg)	-2.824815	T2prep	no
Quantitative flow	no	RL (deg)	-1.097183	+DANTE	no
CENTRA	no	FH (deg)	1.532098	Research prepulse	no
Manual start	no	Free rotatable	no	Diffusion mode	no
Dynamic study	no	<b>POST/PROC</b>		Elastography mode	no
Arterial Spin labeling	no	Preparation phases	auto	Transmit channels	both
<b>POST/PROC</b>		Interactive F0	no	SAR mode	high
Preparation phases	auto	SmartPlan survey	no	B1 mode	default
Interactive F0	no	BO field map	no	SAR Patient data	auto
SmartPlan survey	no	MIP/MPR	no	PNS mode	low
BO field map	no	SWIp	no	Gradient mode	default
MIP/MPR	no	Images	M, no, no, no	SofTone mode	no
SWIp	no	Autoview image	M		
Images	M, no, no, no	Calculated images	no, no, no, no		
Autoview image	M	Reference tissue	White matter		
Calculated images	no, no, no, no	Recon compression	No		
Reference tissue	White matter	Preset window contrast	soft		
Recon compression	No	Reconstruction mode	immediate		
Preset window contrast	soft	Save raw data	no		
Reconstruction mode	immediate	Hardcopy protocol	no		
Save raw data	no	Image filter	no		
Hardcopy protocol	no	Elliptical k-space shutter	default		
Image filter	no				
Elliptical k-space shutter	default				

Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1 | DUAL\_SENSE2 05:00.0

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	05:00.0	Nucleus	H1	Scan type	Imaging
Rel. SNR	1	Uniformity	CLEAR	Scan mode	MS
Act. TR (ms)	3000	FOV AP (mm)	240	technique	SE
Act. TE (ms)	13/100	RL (mm)	240	Modified SE	no
ACQ matrix M x P	256 x 240	FH (mm)	144	Acquisition mode	cartesian
ACQ voxel MPS (mm)	0.94 / 1.00 / 3.00	Voxel size AP (mm)	0.94	Fast Imaging mode	TSE
REC voxel MPS (mm)	0.94 / 0.94 / 3.00	RL (mm)	0.9375	shot mode	multishot
Scan percentage (%)	93.75	Slice thickness (mm)	3	TSE factor	10
Packages	4	Recon voxel size (mm)	0.9375	startup echoes	0
Min. slice gap (mm)	0	Small FOV imaging	no	profile orders	default
Optimal slices	13	Fold-over suppression	no	DRIVE	no
Max. slices	52	Reconstruction matrix	256	ultrashort	no
WFS (pix) / BW (Hz)	1.974 / 219.9	SENSE	yes	fid reduction	default
TSE es / shot (ms)	12.5 / 125	P reduction (RL)	2	Echoes	2
TEeff / TEequiv (ms)	100 / 100	+MultiBand SENSE	no	partial echo	no
Min. TR (ms)	2728	k-t BLAST	no	TE first	shortest
SAR / local torso	< 91 %	Stacks	1	second (ms)	100
Whole body / level	< 2.9 W/kg / 1st level	type	parallel	Flip angle (deg)	90
SED	< 0.9 kJ/kg	slices	48	Refocusing control	no
B1+rms / Coil Power	2.18 uT / 87 %	slice gap	user defined	TR	user defined
Max B1+rms	2.19 uT	gap (mm)	0	(ms)	3000
PNS / level	36 % / normal	slice orientation	transverse	Halfscan	no
dB/dt	37.7 T/s	fold-over direction	RL	Water-fat shift	maximum
Sound Pressure Level (dB)	12.61967	fat shift direction	P	RF Shims	fixed
Boil-off (hPa/h)	1.138171	Minimum number of packages	1	Shim	default
Max Boil-off (hPa/h)	1.138171	Slice scan order	default	mDIXON	no
Boil-off (hPa)	0.09484755	+Slice shuffling	no	Fat suppression	no
<b>MOTION</b>		PlanAlign	no	Grad Rev Fat suppression	no
Cardiac synchronization	no	REST slabs	0	Water suppression	no
Heart rate > 250 bpm	no	Catheter tracking	no	BB pulse	no
Respiratory compensation	no	Interactive positioning	no	MTC	no
Navigator respiratory comp	no	Allow table movement	no	T2prep	no
+RNAV imaging mode	no	<b>OFFC/ANG</b>		+DANTE	no
Flow compensation	no	Stacks	1	Research prepulse	no
Temporal slice spacing	default	Stack Offc. AP (P=+mm)	-18.11262	Zoom imaging	no
Motion smoothing	no	RL (L=+mm)	-3.655661	Diffusion mode	no
NSA	1	FH (H=+mm)	34.79839	Elastography mode	no
<b>DYN/ANG</b>		Ang. AP (deg)	-2.824815	Transmit channels	both
Manual start	no	RL (deg)	-1.097183	SAR mode	high
Dynamic study	no	FH (deg)	1.532098	B1 mode	default
Arterial Spin labeling	no	Free rotatable	no	SAR Patient data	auto
<b>POST/PROC</b>				PNS mode	low
Preparation phases	auto			Gradient mode	default
Interactive F0	no			SoftTone mode	no
SmartPlan survey	no				
B0 field map	no				
MIP/MPR	no				
Images	M, no, no, no				
Autoview image	M				
Calculated images	no, no, no, no				
Reference tissue	Grey matter				
Recon compression	No				
Preset window contrast	soft				
Reconstruction mode	real time				
Save raw data	no				
Hardcopy protocol	no				
Image filter	weak				

Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1 | AX\_FLAIR\_SENSE2 03:36.0

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	03:36.0	Nucleus	H1	Scan type	Imaging
Rel. SNR	1	Uniformity	CLEAR	Scan mode	MS
Act. TR/TI (ms)	9000 / 2500	FOV AP (mm)	240	technique	IR
Act. TE (ms)	125.00	RL (mm)	210	Acquisition mode	cartesian
ACQ matrix M x P	256 x 190	FH (mm)	144	Fast Imaging mode	TSE
ACQ voxel MPS (mm)	0.94 / 1.11 / 3.00	Voxel size AP (mm)	0.94	shot mode	multishot
REC voxel MPS (mm)	0.94 / 0.94 / 3.00	RL (mm)	0.94	TSE factor	19
Scan percentage (%)	84.82143	Slice thickness (mm)	3	startup echoes	0
Packages	4	Recon voxel size (mm)	0.9375	profile order	linear
Min. slice gap (mm)	0.6	Small FOV imaging	no	DRIVE	no
Optimal slices	26	Fold-over suppression	no	ultrashort	yes
Max. slices	52	Reconstruction matrix	256	shift	0
WFS (pix) / BW (Hz)	1.748 / 248.5	SENSE	yes	fid reduction	default
Full flow comp.	yes	P reduction (RL)	2	Echoes	1
TSE es / shot (ms)	12.5 / 238	+MultiBand SENSE	no	partial echo	no
TEff / TEequiv (ms)	125 / 122	k-t BLAST	no	TE	user defined
Min. TR/TI (ms)	8342 / 50	Stacks	1	(ms)	125
SAR / local torso	< 67 %	type	parallel	Refocusing control	yes
Whole body / level	< 2.2 W/kg / 1st level	slices	48	angle (deg)	150
SED	< 0.5 kJ/kg	slice gap	user defined	echo enhancement	no
B1+rms / Coil Power	1.87 uT / 64 %	gap (mm)	0	bright fat reduction	no
Max B1+rms	1.89 uT	slice orientation	transverse	TR	user defined
PNS / level	56 % / normal	fold-over direction	RL	(ms)	9000
dB/dt	44.8 T/s	fat shift direction	P	Halfscan	no
Sound Pressure Level (dB)	21.56663	Minimum number of packages	3	Water-fat shift	maximum
Boil-off (hPa/h)	2.413759	Slice scan order	default	IR delay (ms)	2500
Max Boil-off (hPa/h)	3.807558	+Slice shuffling	no	acquire during delay	yes
Boil-off (hPa)	0.1448256	PlanAlign	no	dual	no
<b>MOTION</b>		REST slabs	1	power	1
Cardiac synchronization	no	type	parallel	RF Shims	fixed
Heart rate > 250 bpm	no	thickness (mm)	53.99319	Shim	default
Respiratory compensation	no	position	feet	mDIXON	no
Navigator respiratory comp	no	gap	default	Fat suppression	SPIR
+RNAV imaging mode	no	power	1	strength	strong
Flow compensation	yes	Catheter tracking	no	frequency offset	default
direction	in-plane	Interactive positioning	no	Grad Rev Fat suppression	no
Motion smoothing	no	Allow table movement	no	Water suppression	no
NSA	1	<b>OFFC/ANG</b>		MTC	no
<b>DYN/ANG</b>		Stacks	1	T2prep	no
Manual start	no	Stack Offc. AP (P=+mm)	-18.11262	+DANTE	no
Dynamic study	no	RL (L=+mm)	-3.655661	Research prepulse	no
Arterial Spin labeling	no	FH (H=+mm)	34.79839	Zoom imaging	no
<b>POST/PROC</b>		Ang. AP (deg)	-2.824815	Diffusion mode	no
Preparation phases	auto	RL (deg)	-1.097183	Elastography mode	no
Interactive F0	no	FH (deg)	1.532098	Transmit channels	both
SmartPlan survey	no	Free rotatable	no	SAR mode	high
B0 field map	no			B1 mode	default
MIP/MPR	no			SAR Patient data	auto
Images	M, no, no, no			PNS mode	high
Autoview image	M			Gradient mode	default
Reference tissue	Grey matter			SofTone mode	no
Recon compression	No				
Preset window contrast	soft				
Reconstruction mode	real time				
Save raw data	no				
Hardcopy protocol	no				
Image filter	weak				

Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56.1 | T2\_FFE\_SENSE2 04:13.5

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	04:13.5	Nucleus	H1	Scan type	Imaging
Rel. SNR	0.9999814	Uniformity	CLEAR	Scan mode	MS
Act. TR/TE (ms)	650 / 21	FOV AP (mm)	240	technique	FFE
ACQ matrix M x P	256 x 256	RL (mm)	240	Contrast enhancement	no
ACQ voxel MPS (mm)	0.94 / 0.94 / 3.00	FH (mm)	144	Acquisition mode	cartesian
REC voxel MPS (mm)	0.94 / 0.94 / 3.00	Voxel size AP (mm)	0.94	Fast Imaging mode	none
Scan percentage (%)	100	RL (mm)	0.9411765	Echoes	1
Packages	3	Slice thickness (mm)	3	partial echo	no
Min. slice gap (mm)	0	Recon voxel size (mm)	0.9375	shifted echo	no
Optimal slices	18	Small FOV imaging	no	TE	in-phase
Max. slices	54	Fold-over suppression	no	(ms)	20.72453
Act. WFS (pix) / BW (Hz)	2,001 / 217.0	Reconstruction matrix	256	Flip angle (deg)	20
Min. WFS (pix) / Max. BW (Hz)	0.907 / 478.7	SENSE	yes	TR	user defined
Min. TR/TE (ms)	549 / 9.4	P reduction (RL)	2	(ms)	650
SAR / local torso	< 63 %	+MultiBand SENSE	no	Halfscan	no
Whole body / level	< 2.0 W/kg / normal	k-t BLAST	no	Water-fat shift	user defined
SED	< 0.5 kJ/kg	Stacks	1	(pixels)	2
B1+rms / Coil Power	1.81 uT / 60 %	type	parallel	RF Shims	fixed
Max B1+rms	1.81 uT	slices	48	Shim	default
PNS / level	14 % / normal	slice gap	user defined	mDIXON	no
dB/dt	6.4 T/s	gap (mm)	0	Fat suppression	no
Sound Pressure Level (dB)	-3.493363	slice orientation	transverse	Water suppression	no
Boil-off (hPa/h)	0.8865204	fold-over direction	RL	MTC	no
Max Boil-off (hPa/h)	0.8865204	fat shift direction	P	Research prepulse	no
Boil-off (hPa)	0.06242581	Minimum number of packages	1	Diffusion mode	no
<b>MOTION</b>		Slice scan order	default	Elastography mode	no
Cardiac synchronization	no	+Slice shuffling	no	Transmit channels	both
Heart rate > 250 bpm	no	PlanAlign	no	SAR mode	high
Respiratory compensation	no	REST slabs	1	B1 mode	default
Navigator respiratory comp	no	shared	no	SAR Patient data	auto
+RNAV imaging mode	no	type	parallel	PNS mode	low
Flow compensation	yes	thickness (mm)	60	Gradient mode	default
+force compact waveforms	no	position	feet	SoftTone mode	yes
Temporal slice spacing	default	gap	default		
fMRI echo stabilisation	no	power	1		
NSA	1	Catheter tracking	no		
<b>DYN/ANG</b>		Interactive positioning	no		
Angio / Contrast enh.	no	Allow table movement	no		
Quantitative flow	no	<b>OFFC/ANG</b>			
Manual start	no	Stacks	1		
Dynamic study	no	Stack Offc. AP (P=+mm)	-18.11262		
Arterial Spin labeling	no	RL (L=+mm)	-3.655661		
<b>POST/PROC</b>		FH (H=+mm)	34.79839		
Preparation phases	auto	Ang. AP (deg)	-2.824815		
Interactive F0	no	RL (deg)	-1.097183		
SmartPlan survey	no	FH (deg)	1.532098		
BO field map	no	Free rotatable	no		
MIP/MPR	no				
SWIp	no				
Images	M, no, no, no				
Autoview image	M				
Calculated images	no, no, no, no				
Reference tissue	Grey matter				
Recon compression	No				
Preset window contrast	soft				
Reconstruction mode	real time				
Save raw data	no				
Hardcopy protocol	no				
Image filter	weak				



Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56:1 | DTI\_32\_iso 05:50.4

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	05:50.4	Nucleus	H1	Scan type	Imaging
Rel. SNR	1.00016	Uniformity	CLEAR	Scan mode	MS
Act. TR (ms)	10012	FOV RL (mm)	256	technique	SE
Act. TE (ms)	107.50	AP (mm)	256	Modified SE	no
ACQ matrix M x P	128 x 126	FH (mm)	140	Acquisition mode	cartesian
ACQ voxel MPS (mm)	2.00 / 2.03 / 2.00	Voxel size RL (mm)	2	Fast Imaging mode	EPI
REC voxel MPS (mm)	2.00 / 2.00 / 2.00	AP (mm)	2	shot mode	single-shot
Scan percentage (%)	98.4375	Slice thickness (mm)	2	Echoes	1
Packages	1	Recon voxel size (mm)	2	partial echo	no
Min. slice gap (mm)	0	Small FOV imaging	no	TE	user defined
Diffusion gradient timing DELTA / delta (ms)	54.0 / 25.0	Fold-over suppression	no	(ms)	107.5
+Diff. Max. Act. Grad. (mT/m)	22.1461	Reconstruction matrix	128	Flip angle (deg)	90
+Diff. Max. Eff. Grad. (mT/m)	22.14624	SENSE	yes	TR	shortest
EPI factor	63	P reduction (AP)	2	Halfscan	no
+EPI echo spacing (ms)	0.68	+MultiBand SENSE	no	Water-fat shift	user defined
WFS (pix) / BW (Hz)	18.816 / 23.1	k-t BLAST	no	(pixels)	23.1
BW in EPI freq. dir. (Hz)	1789.0	Stacks	1	RF Shims	fixed
SAR / local torso	< 28 %	type	parallel	Shim	auto
Whole body / level	< 0.9 W/kg / normal	slices	70	mDIXON	no
SED	< 0.3 kJ/kg	slice gap	user defined	Fat suppression	SPIR
B1+rms / Coil Power	1.21 uT / 27 %	gap (mm)	0	strength	strong
Max B1+rms	1.21 uT	slice orientation	transverse	frequency offset	default
PNS / level	80 % / normal	fold-over direction	AP	Grad Rev Fat suppression	no
dB/dt	91.3 T/s	fat shift direction	P	Water suppression	no
Sound Pressure Level (dB)	21.04232	Minimum number of packages	1	BB pulse	no
Boil-off (hPa/h)	14.27643	Slice scan order	default	MTC	no
Max Boil-off (hPa/h)	14.27677	+Slice shuffling	no	Research prepulse	no
Boil-off (hPa)	1.389714	PlanAlign	no	Diffusion mode	DTI
<b>MOTION</b>		REST slabs	0	sequence	SE
Cardiac synchronization	no	Catheter tracking	no	gradient duration	maximum
Heart rate > 250 bpm	no	Interactive positioning	no	+diffusion time	auto
Respiratory compensation	no	Allow table movement	no	gradient overplus	no
Navigator respiratory comp	no	<b>OFFC/ANG</b>		directional	opt 32
+RNAV imaging mode	no	Stacks	1	resolution	
Flow compensation	no	Stack Offc. AP (P=+mm)	9.605714	nr of b-factors	2
Temporal slice spacing	default	RL (L=+mm)	-0.6080264	b-factor order	ascending
NSA	1	FH (H=+mm)	-1.581959	max b-factor	1000
<b>DYN/ANG</b>		Ang. AP (deg)	4.244757	average high b	no
Manual start	no	RL (deg)	-16.10632	+add noise scan	no
Dynamic study	no	FH (deg)	2.592952	+allow complex	no
dyn stabilization	no	Free rotatable	no	diff.	
Arterial Spin labeling	no			+slider loop order	no
<b>POST/PROC</b>				+image by image	no
Preparation phases	full			scaling	
Interactive F0	no			Elastography mode	no
SmartPlan survey	no			Transmit channels	both
B0 field map	no			SAR mode	high
MIP/MPR	no			B1 mode	default
Images	M, no, no, no			SAR Patient data	auto
Autoview image	M			PNS mode	moderate
Calculated images	no, no, no, no			Gradient mode	maximum
Reference tissue	White matter			SoftTone mode	no
EPI 2D phase correction	no				
Recon compression	No				
Preset window contrast	soft				
Reconstruction mode	immediate				
Save raw data	no				
Hardcopy protocol	no				
Image filter	weak				



Hospital (5) | CDIP\_HUMAN (Master patch -32 canaux) (8) 34:56:1 | DTI\_b0\_AP 00:31.1

INFO PAGE		GEOMETRY		CONTRAST	
Total scan duration	00:31.1	Nucleus	H1	Scan type	Imaging
Rel. SNR	1.000128	Uniformity	CLEAR	Scan mode	MS
Act. TR (ms)	10360	FOV RL (mm)	256	technique	SE
Act. TE (ms)	107.50	AP (mm)	256	Modified SE	no
ACQ matrix M x P	128 x 126	FH (mm)	140	Acquisition mode	cartesian
ACQ voxel MPS (mm)	2.00 / 2.03 / 2.00	Voxel size RL (mm)	2	Fast Imaging mode	EPI
REC voxel MPS (mm)	2.00 / 2.00 / 2.00	AP (mm)	2	shot mode	single-shot
Scan percentage (%)	98.4375	Slice thickness (mm)	2	Echoes	1
Packages	1	Recon voxel size (mm)	2	partial echo	no
Min. slice gap (mm)	0	Small FOV imaging	no	TE	user defined
Diffusion gradient timing DELTA / delta (ms)	54.0 / 20.1	Fold-over suppression	no	(ms)	107.5
+Diff. Max. Act. Grad. (mT/m)	0	Reconstruction matrix	128	Flip angle (deg)	90
+Diff. Max. Eff. Grad. (mT/m)	0	SENSE	yes	TR	shortest
EPI factor	63	P reduction (AP)	2	Halfscan	no
+EPI echo spacing (ms)	0.83	+MultiBand SENSE	no	Water-fat shift	user defined
WFS (pix) / BW (Hz)	23.099 / 18.8	k-t BLAST	no	(pixels)	23.1
BW in EPI freq. dir. (Hz)	1402.4	Stacks	1	RF Shims	fixed
SAR / local torso	< 27 %	type	parallel	Shim	auto
Whole body / level	< 0.9 W/kg / normal	slices	70	mDIXON	no
SED	0.0 kJ/kg	slice gap	user defined	Fat suppression	SPIR
B1+rms / Coil Power	1.19 uT / 26 %	gap (mm)	0	strength	strong
Max B1+rms	1.19 uT	slice orientation	transverse	frequency offset	default
PNS / level	70 % / normal	fold-over direction	AP	Grad Rev Fat suppression	no
dB/dt	94.7 T/s	fat shift direction	A	Water suppression	no
Sound Pressure Level (dB)	20.64277	Minimum number of packages	1	BB pulse	no
Boil-off (hPa/h)	12.43217	Slice scan order	default	MTC	no
Max Boil-off (hPa/h)	12.43445	+Slice shuffling	no	Research prepulse	no
Boil-off (hPa)	0.1073352	PlanAlign	no	Diffusion mode	DTI
<b>MOTION</b>		REST slabs	0	sequence	SE
Cardiac synchronization	no	Catheter tracking	no	gradient duration	maximum
Heart rate > 250 bpm	no	Interactive positioning	no	+diffusion time	auto
Respiratory compensation	no	Allow table movement	no	gradient overplus	no
Navigator respiratory comp	no	<b>OFFC/ANG</b>		directional	low (6)
+RNAV imaging mode	no	Stacks	1	resolution	
Flow compensation	no	Stack Offc. AP (P=+mm)	9.605714	nr of b-factors	1
Temporal slice spacing	default	RL (L=+mm)	-0.6080264	max b-factor	0
NSA	1	FH (H=+mm)	-1.581959	average high b	no
<b>DYN/ANG</b>		Ang. AP (deg)	4.244757	+add noise scan	no
Manual start	no	RL (deg)	-16.10632	+allow complex	no
Dynamic study	no	FH (deg)	2.592952	diff.	
dyn stabilization	no	Free rotatable	no	+slider loop order	no
Arterial Spin labeling	no	<b>POST/PROC</b>		+image by image	no
<b>POST/PROC</b>		Preparation phases	full	scaling	
Preparation phases	full	Interactive F0	no	Elastography mode	no
Interactive F0	no	SmartPlan survey	no	Transmit channels	both
SmartPlan survey	no	B0 field map	no	SAR mode	high
B0 field map	no	MIP/MPR	no	B1 mode	default
MIP/MPR	no	Images	M, no, no, no	SAR Patient data	auto
Images	M, no, no, no	Autoview image	M	PNS mode	moderate
Autoview image	M	Calculated images	no, no, no, no	Gradient mode	maximum
Calculated images	no, no, no, no	Reference tissue	White matter	SoftTone mode	no
Reference tissue	White matter	EPI 2D phase correction	no		
EPI 2D phase correction	no	Recon compression	No		
Recon compression	No	Preset window contrast	soft		
Preset window contrast	soft	Reconstruction mode	immediate		
Reconstruction mode	immediate	Save raw data	no		
Save raw data	no	Hardcopy protocol	no		
Hardcopy protocol	no	Image filter	weak		
Image filter	weak				